

Visage[®] 7

IHE Integration Statement

Visage 7

Information about manufacturer and distribution contacts as well as regulatory status of the product can be found in the User Manual.

Some of the specifications described herein may not be currently available in all countries. Please contact your local sales representative for the most current information.

Visage Imaging is a trademark and/or service mark of ProMedicus. All other products mentioned may be trademarks or registered trademarks of their respective holders.

Visage Imaging believes this information is accurate as of its publication date and is not responsible for any inadvertent errors. The information contained herein is subject to change without notice.

Copyright © Visage Imaging GmbH. All rights reserved.

Document Version 07.00 – Oct 2016

Table of Contents

- 1 Introduction..... 4
 - 1.1 Audience..... 4
 - 1.2 Remarks..... 4
- 2 Specification 5
- 3 About this Document 7
 - 3.1 Document History 7
 - 3.2 Abbreviations and Acronyms 7

1 Introduction

1.1 Audience

This document is intended for hospital staff, health system integrators, software designers or implementers. It is assumed that the reader is familiar with the key concepts of IHE.

1.2 Remarks

This document is the IHE Integration Statement for Visage 7.

IHE, by itself, does not guarantee interoperability. However, the Integration Statement facilitates a first-level validation for interoperability between different applications that are implemented according to the IHE concepts.

This Interface Specification is not intended to replace validation with other IHE equipment to ensure proper exchange of information intended.

The scope of this Interface Specification is to facilitate communication between Visage 7 and other IHE conform systems. The Interface Specification should be read and understood in conjunction with the DICOM and HL7 Standard and the IHE Technical Framework. However, by itself it is not guaranteed to ensure the desired interoperability and a successful interconnectivity.

The user should be aware of the following important issues:

- The comparison of different Interface Specifications is the first step towards assessing interconnectivity between Visage 7 and other IHE conformant equipment.
- Test procedures should be defined to validate the desired level of connectivity.

2 Specification

IHE Integration Statement		Date 2016-09-26
Vendor	Product Name	Version
Visage Imaging GmbH	Visage 7	7.1.9
This product is intended to implement all transactions required in the IHE Technical Framework to support the IHE Integration Profiles, Actors and Options listed below:		

Integration Profiles Implemented	Actors Implemented	Options Implemented
Access to Radiology Information	Image Manager/Archive	None
	Image Display	None
Audit Trail and Node Authentication	Secure Application	None
Consistent Presentation of Images	Image Manager/Archive	None
	Image Display	None
Consistent Time	Time Client	None
Cross-Enterprise Clinical Documents Share (XDS.b)	Document Consumer	None
Digital Breast Tomosynthesis	Image Manager/Archive	None
	Image Display	None
Evidence Documents	Image Manager/Archive	None
Imaging Object Change Management	Image Manager/Archive	None
	Image Display	None
	Change Requester	None
Import Reconciliation Workflow	Image Manager/Archive	None
Mammography Image	Image Manager/Archive	None
	Image Display	None
NM Image	Image Manager/Archive	None
Patient Information Reconciliation	Image Manager/Archive	None
Portable Data for Imaging	Portable Media Creator	None
	Portable Media Importer	None

Integration Profiles Implemented	Actors Implemented	Options Implemented
Scheduled Workflow	Image Manager/Archive	<ul style="list-style-type: none"> - PPS Exception Management - Availability of PPS-Referenced Instances - Performed Work Status Update Receive
	Image Display	None
<u>Scheduled Workflow.b</u>	Image Manager/Archive	<ul style="list-style-type: none"> - PPS Exception Management - Availability of PPS-Referenced Instances - Performed Work Status Update Receive
	Performed Procedure Step Manager	None
Internet address for vendor's IHE information: http://www.visageimaging.com/about/compliance/		
Links to Standards Conformance Statements for the Implementation		
HL7	http://www.visageimaging.com/about/compliance/	
DICOM	http://www.visageimaging.com/about/compliance/	
Links to general information on IHE:		
In North America: http://ihe.net/IHE_Worldwide/	In Europe: http://ihe.net/IHE_Worldwide/	In Japan: http://ihe.net/IHE_Worldwide/

3 About this Document

3.1 Document History

Version	Date	Description
01.00	09-Jan-2008	Release Version for Visage PACS/CS 5.1
03.00	30-Jun-2008	Release Version for Visage PACS/CS 6.0
04.00	18-Nov-2008	Release Version for Visage PACS/CS 6.0
05.00	28-Aug-2009	Release Version for Visage PACS/CS 6.0
06.00	09-Dec-2009	Release Version for Visage 7
06.01	26-Sep-2016	Added results of the 16 th Europe Connectathon in Bochum
06.02	13-Oct-2016	Changed layout of document
06.03	14-Oct-2016	Reviewed new layout, minor improvements
07.00	14-Oct-2016	Approved

3.2 Abbreviations and Acronyms

DICOM	Digital Imaging and Communications in Medicine
HL7	Health Level 7
IHE	Integrating the Healthcare Enterprise
PPS	Performed Procedure Step Manager